

Ingredients:

100 g APESIN DISINFECTANT SPRAY contain: 70 g isopropanol, 0.1 g polyamines. Other ingredients: perfume oil. pH approx. 9.

Activity:

Quick-acting surface disinfectant. Bactericidal, fungicidal, virus-inactivating (hepatitis B, AIDS). Colorless, pleasant fragrance, very kind to the skin. Also suitable for disinfecting surgeons' shoes. Chlorine, aldehyde and phenol-free. Tested and reported on according to the guidelines for the DGHM List (German Society of Hygiene and Microbiology).

Range of application :

For disinfecting equipment, furnishings and surfaces in hospital and general practice. Mattresses, bed frames, trolleys, instrument trays, bath mats and fixtures, waste bins, toilet seats, fixtures, door handles, furniture and work surfaces, handrails, shoes, stockings, etc. as well as all hard-to-reach places can be disinfected. Do not use on materials sensitive to alcohol (e.g. plexiglass).

Directions for use:

Use undiluted. Spray directly onto objects from a distance of 20-30 cm so that the surfaces are completely wetted, or spray onto a cloth and wipe on.

Special precautions:

VbF category B (Order on Facilities for the Storage, Filling and Transporting of Flammable Liquids by Land). Highly flammable. Contains isopropyl alcohol. Must be kept out of the reach of children. Keep away from sources of ignition - no smoking. Do not spray into flames or onto hot objects. Only use in well-ventilated area. In case of fire, use water to extinguish. In accordance with guideline UVV § 7 (Environmental Preventive Regulations), gloves must be worn when handling disinfectants. Application: max. 50 ml / m². Do not spray over large surface areas (max. 2 m²).

This product is intended for industrial use.

Environmental aspects:

This product is subject to the Law on the Environmental Compatibility of Washing and Cleaning Agents and the attached implementing regulations on the biological degradability of raw materials.

Austrian standard S 2100: 55370
